

esthetic.line

## Ceramics

Livento<sup>®</sup> press, Soprano<sup>®</sup> 10, Soprano<sup>®</sup> 14, Soprano<sup>®</sup> Surface,  
Soprano<sup>®</sup> Stains, Shades and Glazes

EN

Summary of Safety and Clinical Performance

English

2

## Preface

This short report based on the current Medical Device Coordination Group document (MDCG 2019-9) is a summary of information regarding safety and clinical performance of the ceramics.

This report is not meant to be used as a practical guide for the application of our products. The current technical data for the correct handling of the products are described in the instructions for use available on [www.cmsa.ch/docs](http://www.cmsa.ch/docs), at the sales representatives or customer service of Cendres+Métaux (CM).

More detailed information on products, the materials used as well as their compositions can be found in the product-specific material data sheets, the product information as well as the instructions for use. These documents as well as numerous clinical cases and scientific publications can be found on the website [www.cmsa.ch/docs](http://www.cmsa.ch/docs) by entering the relevant product name.

**Table of content**

Preface	2
<b>1 Device identification and general information</b>	<b>4</b>
1.1 Device trade name	4
1.2 Manufacturer name and address	4
1.3 Medical device description, basic UDI-DI, classification	4
1.4 Authorised representative	4
1.5 Certificate, single registration number and notified body	4

**PART A**

## Information for user and medical experts

<b>2 Device description</b>	<b>4</b>
2.1 Description of the device group (key functions, design characteristics, materials with body contact)	4
2.2 Intended purpose, medical condition, anatomic locality of application	4
2.3 Indications	5
2.4 Contraindications	5
2.5 Affected articles	5
2.6 Reference and description of variants and previous generations	5
2.7 Description of any accessories	6
2.8 Combination products	6
<b>3 Residual risks and warnings</b>	<b>6</b>
3.1 Residual risks	6
3.2 Side effects	6
3.3 Warnings	6
3.4 Precautions	6
3.5 Other relevant aspect of safety, including a summary of any field safety corrective action	6
<b>4 Summary of clinical evaluation and post-market clinical follow-up</b>	<b>7</b>
4.1 Summary of clinical data related to equivalent device	7
4.2 Summary of clinical data from conducted investigations of the device before the CE-marking	7
4.3 Summary of clinical data from other sources	7
4.4 An overall summary of safety and clinical performance	7
4.5 Ongoing or planned post-market clinical follow-up	7
<b>5 Possible diagnostic or therapeutic alternatives</b>	<b>7</b>
<b>6 Suggested profile and training for users</b>	<b>8</b>
<b>7 Reference to any standards and common specifications (CS) applied</b>	<b>8</b>

**PART B**

## Relevant information for the patient

**PART C**

## Revision history

**PART D**

## List of articles

## 1 Device identification and general information

### 1.1 Device trade name

Livento® press, Soprano® 10, Soprano® 14, Soprano® Surface, Soprano® Stains, Shades and Glazes

### 1.2 Manufacturer name and address

Cendres+Métaux SA  
Rue de Boujean 122  
CH-2501 Biel/Bienne

### 1.3 Medical device description, basic UDI-DI, classification

Medical device	Basic UDI-DI	Class of device according to Annex VIII of the Medical Device Regulation 2017/745
Livento® press various articles	764016651000037E6	Ila
Soprano® 10 various articles	764016651000038E8	Ila
Soprano® 14 various articles		
Soprano® Surface various articles		
Soprano® Stains various articles		
Soprano® Shades various articles		
Soprano® Glazes various articles		
Livento® Soprano® kits	764016651000039EA	Ila

All products including device number are listed in PART D of this document.

### 1.4 Authorised representative

QualRep Services B.V.  
Utrechtseweg 310 – Bldg B42  
NL-6812 AR Arnhem  
The Netherlands

Single registration number: NL-AR-000000537

### 1.5 Certificate, single registration number and notified body

Year when the first certificate was issued: 2018  
Manufacturer's single registration number: CH-MF-000030696  
Notified body: mdc medical device certification GmbH  
Single identification number: 0483

## PART A

### Information for user and medical experts

This Section contains information regarding safety and clinical performance, especially relevant for user and medical experts.

## 2 Device description

### 2.1 Description of the device group (key functions, design characteristics, materials with body contact)

Livento® press are lithium disilicate glass ceramic blanks for the press technique. A blank of 13mm diameter weighs approximately 3g and is for max. 0.3g wax weight. The shades of the pressed blanks are based on the VITA Classic shade system, shade scale A, B, C, and D.

The Soprano® 10 veneering ceramic is a silicate glass ceramic and is extremely well suited for veneers with Livento® lithium disilicate glass ceramics and zirconium oxide. The portfolio includes a balanced manageable range of different ceramic materials such as Margin, Frame, Modifier, Opal, Dentin, Opaque Dentin, enamel and effect materials. The ceramic materials are based on the VITA Classic shade system, shade scale A, B, C, and D.

The metal ceramic Soprano® 14 is based on a silicate glass-ceramic and is stained according to the VITA Classic shade system, shade scale A, B, C, and D. A matched transparency and fluorescence enable the reconstruction of natural teeth as metal-ceramic crowns and bridges on veneering alloys.

Soprano® Surface is a ready-to-use, translucent, opalescent and fluorescent, paste-like glass-ceramic with pigments. It is used to structure and individualize zirconia, lithium disilicate and veneering restorations. Classification according to DIN EN ISO 6872: type 1, Class 1.

Soprano® Flu-Shades, Flu-Stains and Glaze are glass ceramic dental porcelain material. They are intended to finish dental restorations and are offered in a variety of shades.

### 2.2 Intended purpose, medical condition, anatomic locality of application

The products are intended for use for prosthetic restoration and to support procedures in the dental clinic or laboratory. Semi-finished product for replacement of lost tooth hard structure in porcelain fused to metal / porcelain fused to zirconia (PFM / PFZ) or all ceramic technology.

### 2.3 Indications

#### **Livento® press / Soprano® 10:**

- Veneers
- Inlays and onlays
- Partial crowns
- Anterior and posterior tooth crowns
- Hybrid abutment crown
- 3 pontic bridge in anterior tooth region
- 3 pontic bridge in the premolar region up to max. 2nd premolar as terminal abutment

#### **Soprano® 14:**

- Ceramic veneering of dental metal alloys (non-precious metals and precious metals) with a thermal coefficient of expansion of  $13.5\text{-}15 \times 10^{-6} \text{K}^{-1}$  (25-500°C)

#### **Soprano® Surface:**

Use on the following framework materials:

- Fully anatomically fabricated crowns and bridges in tetragonal stabilized zirconia (Y-TCP) with a thermal expansion of approximately  $10.6 \times 10^{-6} \text{K}^{-1}$  (25-500°C).
- Crowns and bridges in tetragonal stabilized zirconia (Y-TZP) with a thermal expansion of approximately  $10.6 \times 10^{-4} \text{K}^{-1}$  (25-500°C), veneered with zirconia oxide ceramic, for example Soprano® 10.
- Monolithic lithium disilicate glass ceramic materials with a thermal expansion of approximately  $10.0 \times 10^{-4} \text{K}^{-1}$  (25-500°C), for example Livento® press.
- Lithium disilicate glass ceramic materials with a thermal expansion of approximately  $10.0 \times 10^{-4} \text{K}^{-1}$  (25-500°C), for example Livento® press veneered with Soprano® 10.
- PFM restorations, for example veneered with Soprano® 14.

#### **Soprano® Stain & Glaze**

- Monolithic or zirconia frameworks veneered with Soprano® 10.
- Monolithic or lithium disilicate frameworks veneered with Soprano® 10 (Livento® press)
- Metal ceramics (Soprano® 14)
- Paste ceramics (Soprano® Surface)

### 2.4 Contraindications

#### **Livento® press / Soprano® 10:**

- Free-end bridges and extension pontics
- 3-pontic bridge in molar region
- Patients with profoundly reduced residual dentition
- Falling below the necessary connector and minimum strengths
- Combination with materials outside the described Livento® press and Soprano® 10 product system and/or with materials from third party suppliers
- As a matter of principle, all indications not listed are to be regarded as contraindications

#### **Soprano® 14:**

- Veneering of non-indexed framework materials
- Combination with other metal ceramics or veneering ceramics for metal-free restorations
- Use of other liquids
- Mixing of powdery materials (e.g. dentin, incisal, etc.) with paste-like materials (e.g. shades, stains)
- Non-anatomically reduced metal frameworks and sharp edges and corners

#### **Soprano® Surface:**

- Veneering of non-indexed framework materials
- Mixing with other ceramics (powders, pastes)
- Inlaying with other ceramics (powders, pastes)
- Use of other liquids

#### **Soprano® Stain & Glaze**

- Combination with materials outside the described Livento® press and Soprano® 10 product system and/or with materials from third party suppliers
- As a matter of principle, all applications not listed are to be regarded as contraindications

#### **All products:**

- Patients who are unable to keep the regularly required check-up appointments for health reasons.
- Patients with bruxism or other para-functional habits
- Patients with allergies to materials used in the product
- Existing clinical picture in the patient's mouth does not permit the correct application of the products.

### 2.5 Affected articles

N/A

### 2.6 Reference and description of variants and previous generations

Variants	
N/A	N/A
Previous generations (inclusive justification of change)	
N/A	N/A

**2.7 Description of any accessories****Available components**

08055277	Soprano® Modeling Liquid
08055014	Livento® invest
083739	Livento® invest Liquid
08055017	Livento® press Disposable Pressplunger (Diameter 13 mm)
08055284	Soprano® Frame Liquid
08057516	Soprano® Margin Liquid
08057665	Soprano® Modeling Liquid
08057666	Soprano® Margin Liquid
08057915	Soprano® Modeling Liquid

**Auxiliary parts**

N/A	N/A
-----	-----

**Auxiliary instruments**

N/A	N/A
-----	-----

**2.8 Combination products**

The following products are combined according to the instruction for use with the following materials:

- Soprano® 10 with zirconium oxide and lithium disilicate
- Soprano® 14 with metal alloys (precious and non-precious metals)
- Soprano® surface with zirconium oxide, lithium disilicate, Soprano® 10 and Soprano® 14
- Soprano® Stain and Glaze with zirconium oxide, lithium disilicate, Soprano® 10, Soprano® 14 and Soprano® Surface

**3 Residual risks and warnings****3.1 Residual risks**

The highest residual risk is debonding or fracture of the dental ceramic. Under rare circumstances, swallowing or aspiration of fractured parts may occur and could have serious consequences, but have never been reported to CM until now.

**3.2 Side effects**

This product may not be used in patients with allergies to one or more elements of the product materials. In patients with suspected allergy to one or more elements of the materials, this product may only be used following allergological clarification and proof of non-existence of an allergy.

Auxiliary instruments and products made of steel may contain nickel.

No known side effects if applied as intended.

**3.3 Warnings**

Magnetic resonance (MR) environment

The device has not been evaluated for safety and compatibility in the MR environment.

The product has not been tested for heating or migration in the MR environment.

**3.4 Precautions**

- The mechanical cleaning with a toothbrush and toothpaste may lead to premature wear.
- When grinding, wear protective goggles and a dust mask and use a suction unit.
- Only original tools and parts may be used for this work. For information and additional details, please contact your Cendres+Métaux SA representative.
- The product components are supplied non-sterile.
- Secure parts against aspiration.
- Before any procedure, ensure that all required components are available in sufficient quantity.
- For your safety, always wear suitable protective clothing.

**3.5 Other relevant aspect of safety, including a summary of any field safety corrective action**

N/A

## 4 Summary of clinical evaluation and post-market clinical follow-up

### 4.1 Summary of clinical data related to equivalent device

- IPS e.max Press from Ivoclar Vivadent AG (Schaan, Liechtenstein) and Cergo Kiss from DeguDent GmbH (Hanau, Germany) is similar to Livento® press
- IPS InLine from Ivoclar Vivadent AG (Schaan, Liechtenstein) is similar to Soprano® 14
- Creation CC from KLEMA Dentalprodukte GmbH (Meiningen, Austria) and CREATION TI from KLEMA Dentalprodukte GmbH (Meiningen, Austria) is similar to Soprano® 14
- IPS e.max ceram from Ivoclar Vivadent AG (Schaan, Liechtenstein) is similar to Soprano® 10
- VITA VM9 from VITA Zahnfabrik H. Rauter GmbH & Co. KG (Bad Säckingen, Germany) is similar to Soprano® 10
- IPS Ivocolor from Ivoclar Vivadent AG (Schaan, Liechtenstein) is similar to Soprano® Stains and Glazes

Detailed instructions for use, brochure and scientific documentation are available for the similar devices, providing detailed information on the clinical application as well as technical specifications, such as shades, flexural strength, baking temperature and materials. These documents include all necessary information to show clinical, technical, and biological similarity, additional data and documents would not provide further meaningful information for a proper scientific justification of equivalency.

### 4.2 Summary of clinical data from conducted investigations of the device before the CE-marking

N/A

### 4.3 Summary of clinical data from other sources

N/A

### 4.4 An overall summary of safety and clinical performance

Based on the clinical evaluation provided by esthetic ceram, it can be concluded that the potential risks of the various dental glass ceramics, i.e. Livento® press, Soprano® 10, Soprano® 14, Soprano Surface and Soprano® Stains, Shades and Glazes, are acceptable residual risks for the patient and the user. The main risks are described in detail in the scientific literature. Based on the relevant scientific literature, it can be concluded that the risks of the use of dental glass ceramics are well-documented in the published literature, thus being known to dental technicians and dental physicians. When complying with all warnings and precautions, these various dental glass ceramic materials under evaluation have an acceptable benefit-risk ratio.

Presented post-market surveillance (PMS) data correspond to a low complaint rate. Furthermore, the products covered in this clinical evaluation have been in clinical use for several years and are well-characterized raw materials for use in dental restoration applied by dental technicians. The source and extent of clinical data and experience generation is justified given the nature and the risk characteristic of the products. Therefore, it was decided that further dedicated scientific post-market clinical follow-up (PMCF) measures beyond regular complaint and incident statistics are not required.

In conclusion, the various dental glass ceramics under evaluation could be shown to be in compliance with the General Safety and Performance Requirements. However, it is important to notice that the decision to use the devices with their potential risks and complications for dental restorations in a particular patient is the essential responsibility of the medical operator. The clinical user's judgment, on the other hand, must be based on current knowledge in medical science and the specific situation of the patient.

### 4.5 Ongoing or planned post-market clinical follow-up

N/A, justification see Section 4.4 an overall summary of safety and clinical performance.

## 5 Possible diagnostic or therapeutic alternatives

Depicted from the clinical evaluation report (CER) by Novineon, July 2020 (see also amendment CER ceramics):

Endodontic treatment involves removal of the dental pulp and its replacement by a root canal filling. Restoration of root filled teeth can be challenging due to structural differences between vital and non-vital root-filled teeth. Direct restoration involves placement of a restorative material e.g. amalgam or composite, directly into the tooth. Indirect restorations consist of cast metal or ceramic crowns.

According to a Cochrane review by Sequeira-Byron and colleagues, there is insufficient evidence to assess the effects of crowns compared to conventional fillings for the restoration of root-filled teeth. Thus, dental physicians should continue to base decisions about how to restore root-filled teeth on their own clinical experience, whilst taking into consideration the individual circumstances and preferences of their patients.

Preformed crowns for primary molars are recommended for teeth with pulp treatment, with a high level of decay or with structural impairment. Such crowns are made of a variety of materials. For example, stainless steel, stainless steel with veneer or ceramic material for better aesthetic results. A Cochrane analysis by Innes and colleagues aimed on the clinical effectiveness and safety of all types of preformed crowns for restoring primary teeth compared with conventional filling materials (such as amalgam, composite, glass ionomer, resin modified glass ionomer and compomers), other types of crowns or methods of crown placement, non-restorative caries treatment or no treatment. The analysis revealed that teeth restored with preformed crowns are less likely to develop problems (e.g. abscess) or cause pain in the long term, compared to fillings. One subgroup analysis compared stainless steel crowns to an additional application of white veneers. There was only one split-mouth study available. Thus, no general conclusion could be made.

Also, a systematic review on clinical effectiveness of all types of aesthetic preformed crowns for restoring primary teeth compared with conventional filling materials or other types of crowns performed by Aiem resulted in the notion that aesthetic Zircon crowns gave significantly better results than the others with regard to gingival health and crown fractures in a pediatric patient population (Aiem E et al., 2016).

A systematic review on zirconia fixed dental prostheses revealed that, at least, short term clinical data suggest that zirconia-based fixed dental prostheses may serve as an alternative to metal ceramic fixed dental prostheses in the anterior and posterior dentition (Raigrodski AJ et al., 2012). A correct bonding strategy for zirconia also seems to be essential for long-term survival. The ability of zirconia to bond to tooth structure has been and is a subject of debate because of its polycrystalline nature (Quigley NP et al., 2020).

## 6 Suggested profile and training for users

The expertise of a professional dentist or dental technician is required. The current instructions for use must be available at all times and be completely read and understood before the first application. The manufacturing work and its maintenance must be carried out by qualified specialists.

## 7 Reference to any standards and common specifications (CS) applied

- EN ISO 9001: Quality management systems – Requirements
  - EN ISO 14001: Environmental Management System – Requirements with guidance for use
  - EN ISO 13485: Medical devices, Quality management systems, Requirements for Regulatory purposes  
Corrigendum AC
  - SN EN ISO 14971: Medical devices - Application of risk management to medical devices
  - SN EN ISO 15223-1: Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements
  - SN EN ISO 20417: Medical devices – Information to be supplied by the manufacturer
  - SN EN ISO/IEC 17050-1: Conformity assessment - Supplier's declaration of conformity - Part 1: General requirements
  - SN EN ISO/IEC 17050-2: Conformity assessment - Supplier's declaration of conformity - Part 2: Supporting documentation
  - IEC 62366-1: Medical devices - Part 1: Application of usability engineering to medical devices, Technical Corrigendum and Amendment 1
  - ISO 9693: Dentistry - Compatibility testing for metal-ceramic and ceramic-ceramic systems
  - SN EN ISO 6872: Dentistry - Ceramic materials and Amendment A1
  - ISO 7405: Dentistry - Evaluation of biocompatibility of medical devices used in dentistry
  - SN EN ISO 10993-1: Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management process
  - SN EN ISO 10993-5: Biological evaluation of medical devices - Part 5: Tests for in vitro cytotoxicity
  - SN EN ISO 10993-17: Biological evaluation of medical devices - Part 17: Establishment of allowable limits for leachable substances
  - SN EN ISO 10993-18: Biological evaluation of medical devices - Part 18: Chemical characterization of materials
  - Standard list of SSCP of esthetic ceram, which includes identical products
- No common specification (CS) yet available

## PART B

### Relevant information for the patient

This Section contains information regarding safety and clinical performance, especially relevant for patients.

N/A

## PART C

### Revision history

Edition	Issue date	Change description	Edition validated by notified body
1.0	02.2022	First version of SSCP for ceramics	<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No Language used: english
2.0	03.2023	Second version of SSCP, new layout FO 7.255 V02, new authorized representative, SRNs now available, EMDN code listed	<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No Language used: english
3.0	03.2024	Third version of SSCP with strictly minor editorial modifications; annual update: adaption of version number and edition, in Preface MDCG document mentioned, Soprano® Stain & Glaze in indications and contraindications added, standard list completed, EMDN code added, and 2 product descriptions corrected.	<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No Language used: english
4.0	18.02.2025	Annual update according to MDR with strictly minor editorial modifications including mdc-deviation (ID 5129: complete version/edition date included in document, together with reference number on front page, and actualization of list of articles).	<input type="checkbox"/> Yes <input type="checkbox"/> No Language used: english

**PART D**

## List of articles

EMDN code:  
ceramics: Q010699

Device number	Product name	Class
08058660	Soprano® Surface Gingiva blood	Ila
08058661	Soprano® Surface Gingiva root	Ila
08058662	Soprano® Stain blood	Ila
08058738	Soprano® Surface Gingiva Intro Kit	Ila
08058739	Soprano® Surface Gingiva Master Kit	Ila
08058740	Soprano® Surface Gingiva dark-intense	Ila
08058741	Soprano® Surface Gingiva bone	Ila
08058742	Soprano® 10 Gingiva light	Ila
08058743	Soprano® 10 Gingiva light	Ila
08058744	Soprano® 10 Gingiva light	Ila
08058745	Soprano® 10 Gingiva dark	Ila
08058746	Soprano® 10 Gingiva dark	Ila
08058747	Soprano® 10 Gingiva dark	Ila
08058748	Soprano® 10 Gingiva dark-intense	Ila
08058749	Soprano® 10 Gingiva dark-intense	Ila
08058750	Soprano® 10 Gingiva dark-intense	Ila
08058751	Soprano® 10 Gingiva orange	Ila
08058752	Soprano® 10 Gingiva orange	Ila
08058753	Soprano® 10 Gingiva violet	Ila
08058754	Soprano® 10 Gingiva violet	Ila
08058755	Soprano® 10 Gingiva brown	Ila
08058756	Soprano® 10 Gingiva brown	Ila
08058757	Soprano® 10 Gingiva blood	Ila
08058758	Soprano® 10 Gingiva blood	Ila
08058759	Soprano® 10 Gingiva bone	Ila
08058760	Soprano® 10 Dentine A1	Ila
08058761	Soprano® 10 Dentine A2	Ila
08058762	Soprano® 10 Dentine A3	Ila
08058763	Soprano® 10 Dentine A3	Ila
08058764	Soprano® 10 Dentine A3,5	Ila
08058765	Soprano® 10 Dentine A3,5	Ila
08058766	Soprano® 10 Dentine B1	Ila
08058767	Soprano® 10 Dentine B2	Ila
08058768	Soprano® 10 Dentine B2	Ila
08058769	Soprano® 14 Gingiva dark-intense	Ila
08058770	Soprano® 14 Gingiva dark-intense	Ila
08058771	Soprano® 14 Gingiva brown	Ila
08058772	Soprano® 14 Gingiva brown	Ila
08058773	Soprano® 14 Gingiva blood	Ila
08058774	Soprano® Flu-Glaze Paste	Ila
08058775	Soprano® POWDER Flu-Glaze	Ila
08055137	Livento® press LT A1	Ila
08055138	Livento® press ET 1	Ila
08055139	Livento® press ET 2	Ila
08055140	Livento® press ET 3	Ila
08055141	Livento® press HO 1	Ila
08055142	Livento® press LT A0	Ila
08055143	Livento® press LT A2	Ila
08055144	Livento® press LT A3	Ila
08055145	Livento® press LT A3,5	Ila
08055146	Livento® press LT A4	Ila
08055147	Livento® press LT B1	Ila
08055148	Livento® press LT B2	Ila
08055149	Livento® press LT B3	Ila
08055150	Livento® press LT B4	Ila
08055151	Livento® press LT C1	Ila
08055152	Livento® press LT C2	Ila

Device number	Product name	Class
08055153	Livento® press LT C3	Ila
08055154	Livento® press LT C4	Ila
08055155	Livento® press LT D2	Ila
08055156	Livento® press LT D3	Ila
08055157	Livento® press LT D4	Ila
08055158	Livento® press MT A0	Ila
08055159	Livento® press MT A1	Ila
08055160	Livento® press MT A2	Ila
08055161	Livento® press MT A3	Ila
08055162	Livento® press MT B1	Ila
08055163	Livento® press MT B2	Ila
08055164	Livento® press MT C1	Ila
08055165	Livento® press MT C2	Ila
08055166	Livento® press MT D2	Ila
08055167	Livento® press Opal 1	Ila
08055168	Livento® press Opal 2	Ila
08056942	Livento® press Bleach 1 MT	Ila
08056943	Livento® press Bleach 2 MT	Ila
08056944	Livento® press Bleach 3 MT	Ila
08056945	Livento® press Bleach 4 MT	Ila
08056946	Livento® press Bleach 1 LT	Ila
08056947	Livento® press Bleach 2 LT	Ila
08056948	Livento® press Bleach 3 LT	Ila
08056949	Livento® press Bleach 4 LT	Ila
08056950	Livento® press Bleach ET	Ila
08056951	Livento® press Bleach HO 0	Ila
08056954	Livento® press Bleach MO 0	Ila
08056955	Livento® press Bleach MO 1	Ila
08057846	Livento® press LT A1	Ila
08057847	Livento® press ET 1	Ila
08057848	Livento® press ET 2	Ila
08057849	Livento® press ET 3	Ila
08057850	Livento® press HO 1	Ila
08057851	Livento® press LT A0	Ila
08057852	Livento® press LT A2	Ila
08057853	Livento® press LT A3	Ila
08057854	Livento® press LT A3,5	Ila
08057855	Livento® press LT A4	Ila
08057856	Livento® press LT B1	Ila
08057857	Livento® press LT B2	Ila
08057858	Livento® press LT B3	Ila
08057859	Livento® press LT B4	Ila
08057867	Livento® press MT A0	Ila
08057868	Livento® press MT A1	Ila
08057869	Livento® press MT A2	Ila
08057870	Livento® press MT A3	Ila
08057871	Livento® press MT B1	Ila
08057872	Livento® press MT B2	Ila
08057886	Livento® press Bleach 1 LT	Ila
08057887	Livento® press Bleach 2 LT	Ila
08057888	Livento® press Bleach 3 LT	Ila
08057889	Livento® press Bleach 4 LT	Ila
08057890	Livento® press Bleach ET	Ila
08057891	Livento® press Bleach HO 0	Ila
08057892	Livento® press Bleach MO 0	Ila
08057893	Livento® press Bleach MO 1	Ila
08055173	Soprano® 10 Margin A2	Ila
08055176	Soprano® 10 Opaque Dentine A2	Ila
08055177	Soprano® 10 Dentine A2	Ila
08055178	Soprano® 10 Mamelon cream	Ila
08055179	Soprano® 10 Enamel Effect light-blue	Ila
08055180	Soprano® 10 Opal Effect	Ila
08055185	Soprano® 10 Frame Modifier A	Ila
08055186	Soprano® 10 Frame Modifier B	Ila

Device number	Product name	Class
08055187	Soprano® 10 Margin A0	Ila
08055188	Soprano® 10 Margin A4	Ila
08055189	Soprano® 10 Margin B2	Ila
08055190	Soprano® 10 Margin C2	Ila
08055191	Soprano® 10 Opaque Dentine A0	Ila
08055192	Soprano® 10 Opaque Dentine A1	Ila
08055193	Soprano® 10 Opaque Dentine A3	Ila
08055194	Soprano® 10 Opaque Dentine A3,5	Ila
08055195	Soprano® 10 Opaque Dentine A4	Ila
08055196	Soprano® 10 Opaque Dentine B0	Ila
08055197	Soprano® 10 Opaque Dentine B1	Ila
08055198	Soprano® 10 Opaque Dentine B2	Ila
08055199	Soprano® 10 Opaque Dentine B3	Ila
08055200	Soprano® 10 Opaque Dentine B4	Ila
08055201	Soprano® 10 Opaque Dentine orange	Ila
08055202	Soprano® 10 Opaque Dentine brown	Ila
08055203	Soprano® 10 Opaque Dentine yellow	Ila
08055204	Soprano® 10 Dentine A0	Ila
08055205	Soprano® 10 Dentine A1	Ila
08055206	Soprano® 10 Dentine A3	Ila
08055207	Soprano® 10 Dentine A3,5	Ila
08055208	Soprano® 10 Dentine A4	Ila
08055209	Soprano® 10 Dentine B0	Ila
08055210	Soprano® 10 Dentine B1	Ila
08055211	Soprano® 10 Dentine B2	Ila
08055212	Soprano® 10 Dentine B3	Ila
08055213	Soprano® 10 Dentine B4	Ila
08055214	Soprano® 10 Enamel E1	Ila
08055215	Soprano® 10 Enamel E2	Ila
08055216	Soprano® 10 Enamel E3	Ila
08055217	Soprano® 10 Enamel E4	Ila
08055218	Soprano® 10 Enamel Clear	Ila
08055219	Soprano® 10 Opal Intensive 2	Ila
08055220	Soprano® 10 Opal Intensive 4	Ila
08055221	Soprano® 10 Opal Clear	Ila
08055222	Soprano® 10 Enamel Effect clear-white	Ila
08055223	Soprano® 10 Enamel Effect sun	Ila
08055224	Soprano® 10 Enamel Effect apricot	Ila
08055225	Soprano® 10 Enamel Effect amber	Ila
08055226	Soprano® 10 Enamel Effect gray	Ila
08055227	Soprano® 10 Enamel Enhancer orange	Ila
08055228	Soprano® 10 Mamelon mango	Ila
08055229	Soprano® 10 Repair	Ila
08055230	Soprano® 10 Frame Modifier D	Ila
08055231	Soprano® 10 Opaque Dentine C1	Ila
08055232	Soprano® 10 Opaque Dentine C2	Ila
08055233	Soprano® 10 Opaque Dentine C3	Ila
08055234	Soprano® 10 Opaque Dentine C4	Ila
08055235	Soprano® 10 Opaque Dentine D2	Ila
08055236	Soprano® 10 Opaque Dentine D3	Ila
08055237	Soprano® 10 Opaque Dentine D4	Ila
08055238	Soprano® 10 Dentine C1	Ila
08055239	Soprano® 10 Dentine C2	Ila
08055240	Soprano® 10 Dentine C3	Ila
08055241	Soprano® 10 Dentine C4	Ila
08055242	Soprano® 10 Dentine D2	Ila
08055243	Soprano® 10 Dentine D3	Ila
08055244	Soprano® 10 Dentine D4	Ila
08055260	Soprano® 10 Gingiva dark	Ila
08055261	Soprano® 10 Gingiva violet	Ila
08055264	Soprano® 10 Bleach Opaque Dentine	Ila
08055268	Soprano® 10 Enamel E0	Ila
08055269	Soprano® 10 Opal Intensive 1	Ila
08055270	Soprano® 10 Opal Intensive 3	Ila

Device number	Product name	Class
08055271	Soprano® 10 Enamel Effect Enh. amber	Ila
08055272	Soprano® 10 Enamel Effect Enh. dark-gray	Ila
08055273	Soprano® 10 Enamel Effect Enh. pink	Ila
08055274	Soprano® 10 Enamel Effect Enh. blue	Ila
08055275	Soprano® 10 Cuspid	Ila
08055276	Soprano® 10 Fossa	Ila
08055326	Soprano® 10 Frame Modifier C	Ila
08056953	Soprano® 10 Bleach Opaque Dentine	Ila
08056956	Soprano® 10 Bleach Dentine 1	Ila
08056957	Soprano® 10 Bleach Dentine 2	Ila
08056958	Soprano® 10 Bleach Dentine 3	Ila
08056959	Soprano® 10 Bleach Dentine 4	Ila
08056960	Soprano® 10 Cuspid	Ila
08056961	Soprano® 10 Bleach Enamel 1	Ila
08056962	Soprano® 10 Bleach Enamel 2	Ila
08056963	Soprano® 10 Enamel EO	Ila
08056964	Soprano® 10 Bleach Transpa-white	Ila
08057760	Soprano® 10 Dentine A0	Ila
08057761	Soprano® 10 Bleach Dentine 4	Ila
08057762	Soprano® 10 Bleach Dentine 4	Ila
08057763	Soprano® 10 Dentine A1	Ila
08057764	Soprano® 10 Dentine A2	Ila
08057765	Soprano® 10 Dentine B0	Ila
08057766	Soprano® 10 Bleach Dentine 3	Ila
08057767	Soprano® 10 Bleach Dentine 3	Ila
08057768	Soprano® 10 Dentine B1	Ila
08057769	Soprano® 10 Enamel E1	Ila
08057770	Soprano® 10 Enamel E1	Ila
08057771	Soprano® 10 Enamel E2	Ila
08057772	Soprano® 10 Enamel E2	Ila
08057773	Soprano® 10 Frame Modifier pink	Ila
08057774	Soprano® 10 Bleach Dentine 1	Ila
08057775	Soprano® 10 Bleach Dentine 1	Ila
08057776	Soprano® 10 Bleach Dentine 2	Ila
08057777	Soprano® 10 Bleach Dentine 2	Ila
08057778	Soprano® 10 Bleach Enamel 1	Ila
08057779	Soprano® 10 Bleach Enamel 1	Ila
08057780	Soprano® 10 Bleach Enamel 2	Ila
08057781	Soprano® 10 Bleach Enamel 2	Ila
08057782	Soprano® 10 Enamel EO	Ila
08057783	Soprano® 10 Enamel EO	Ila
08057909	Soprano® 10 Opaque Dentine A2	Ila
08057910	Soprano® 10 Dentine A2	Ila
08057911	Soprano® 10 Enamel E2	Ila
08057912	Soprano® 10 Opal Effect	Ila
08057913	Soprano® 10 Enamel Effect light-blue	Ila
08057914	Soprano® 10 Mamelon cream	Ila
08058200	Soprano® 10 Gingiva light	Ila
08058201	Soprano® 10 Gingiva orange	Ila
08055181	Soprano® Flu-Shade A	Ila
08055182	Soprano® Flu-Stain dark-blue	Ila
08055183	Soprano® Flu-Stain brown	Ila
08055184	Soprano® Glaze Paste	Ila
08055245	Soprano® Flu-Shade B	Ila
08055246	Soprano® Flu-Shade C	Ila
08055247	Soprano® Flu-Shade D	Ila
08055248	Soprano® Flu-Stain white	Ila
08055249	Soprano® Flu-Stain yellow	Ila
08055250	Soprano® Flu-Stain orange	Ila
08055251	Soprano® Flu-Stain intense-orange	Ila
08055252	Soprano® Flu-Stain navy-blue	Ila
08055253	Soprano® Flu-Stain black	Ila
08055254	Soprano® Flu-Stain gray	Ila
08055255	Soprano® Stain pink	Ila

Device number	Product name	Class
08057511	Soprano® POWDER Flu-Shade A	Ila
08057512	Soprano® POWDER Flu-Stain brown	Ila
08057513	Soprano® POWDER Flu-Stain dark-blue	Ila
08057667	Soprano® POWDER Glaze	Ila
08057745	Soprano® POWDER Stain & Glaze Universal Kit	Ila
08057746	Soprano® POWDER Flu-Shade B	Ila
08057747	Soprano® POWDER Flu-Shade C	Ila
08057748	Soprano® POWDER Flu-Shade D	Ila
08057749	Soprano® POWDER Flu-Stain white	Ila
08057750	Soprano® POWDER Flu-Stain yellow	Ila
08057751	Soprano® POWDER Flu-Stain orange	Ila
08057752	Soprano® POWDER Flu-Stain intense-orange	Ila
08057753	Soprano® POWDER Flu-Stain navy-blue	Ila
08057754	Soprano® POWDER Flu-Stain black	Ila
08057755	Soprano® POWDER Flu-Stain gray	Ila
08057756	Soprano® POWDER Stain pink	Ila
08057926	Soprano® POWDER Glaze	Ila
08058233	Soprano® Stain red	Ila
08058234	Soprano® Stain dark-blue	Ila
08058235	Soprano® Stain red-brown	Ila
08058236	Soprano® Stain orange	Ila
08058238	Soprano® Flu-Shade Bleach 1 - 2	Ila
08058239	Soprano® Flu-Shade Bleach 3 - 4	Ila
08058240	Soprano® PASTE Stain&Glaze Universal Kit	Ila
08058241	Soprano® Flu-Stain snow	Ila
08058242	Soprano® Flu-Stain cotton	Ila
08058243	Soprano® Flu-Stain cream	Ila
08058244	Soprano® Flu-Stain mango	Ila
08058245	Soprano® Flu-Stain champagne	Ila
08058246	Soprano® Flu-Stain olive	Ila
08058247	Soprano® Flu-Stain shadow	Ila
08058248	Soprano® Flu-Stain red-brown	Ila
08055072	Livento® press Soprano® 10 Intro Kit A2	Ila
08055073	Livento® press Soprano® 10 Shade A-B Kit	Ila
08055074	Livento® press Soprano® 10 Shade C-D Kit	Ila
08055077	Livento® press Soprano® 10 Advanced Kit	Ila
08056952	Livento® press Soprano® 10 Bleaching Extended Kit	Ila
08058206	Soprano® Surface Kit	Ila
08058207	Soprano® Surface Base 1	Ila
08058208	Soprano® Surface Base 3	Ila
08058209	Soprano® Surface Base 5	Ila
08058210	Soprano® Surface Mamelon mango	Ila
08058211	Soprano® Surface Mamelon cream	Ila
08058212	Soprano® Surface Mamelon honey	Ila
08058213	Soprano® Surface Mamelon salmon	Ila
08058214	Soprano® Surface Fossa	Ila
08058215	Soprano® Surface Cuspid	Ila
08058216	Soprano® Surface Enamel Effect light-blue	Ila
08058217	Soprano® Surface Enamel Effect pink	Ila
08058218	Soprano® Surface Enamel Effect int.-orange	Ila
08058219	Soprano® Surface Enamel Effect sun	Ila
08058220	Soprano® Surface Enamel V1	Ila
08058221	Soprano® Surface Enamel V2	Ila
08058222	Soprano® Surface Enamel V3	Ila
08058223	Soprano® Surface Opal	Ila
08058224	Soprano® Surface Incisal	Ila
08058225	Soprano® Surface Clear	Ila
08058226	Soprano® Surface Fluid	Ila
08058227	Soprano® Surface Gingiva Kit	Ila
08058228	Soprano® Surface Gingiva light	Ila
08058229	Soprano® Surface Gingiva violet	Ila
08058230	Soprano® Surface Gingiva dark	Ila
08058231	Soprano® Surface Gingiva orange	Ila
08058232	Soprano® Surface Gingiva brown	Ila

Device number	Product name	Class
08058237	Soprano® Surface Bleaching Kit	Ila
08057500	Soprano® 14 Intro Kit A2	Ila
08057501	Soprano® 14 Bonder Paste EMF Alloys	Ila
08057502	Soprano® 14 Paste Opaque A2	Ila
08057503	Soprano® 14 Margin A2	Ila
08057504	Soprano® 14 Opaque Dentine A2	Ila
08057505	Soprano® 14 Dentine A2	Ila
08057506	Soprano® 14 Enamel E2	Ila
08057507	Soprano® 14 Opal Effect	Ila
08057508	Soprano® 14 Enamel Effect light-blue	Ila
08057509	Soprano® 14 Enamel Effect clear-white	Ila
08057510	Soprano® 14 Mamelon cream	Ila
08057517	Soprano® 14 Shade A-B Kit	Ila
08057520	Soprano® 14 Paste Opaque A0	Ila
08057521	Soprano® 14 Paste Opaque A1	Ila
08057522	Soprano® 14 Paste Opaque A3	Ila
08057523	Soprano® 14 Paste Opaque A3,5	Ila
08057524	Soprano® 14 Paste Opaque A4	Ila
08057525	Soprano® 14 Paste Opaque B0	Ila
08057526	Soprano® 14 Paste Opaque B1	Ila
08057527	Soprano® 14 Paste Opaque B2	Ila
08057528	Soprano® 14 Paste Opaque B3	Ila
08057529	Soprano® 14 Paste Opaque B4	Ila
08057530	Soprano® 14 Margin A0	Ila
08057531	Soprano® 14 Margin A1	Ila
08057533	Soprano® 14 Margin A3	Ila
08057534	Soprano® 14 Margin A3,5	Ila
08057535	Soprano® 14 Margin A4	Ila
08057536	Soprano® 14 Margin B0	Ila
08057537	Soprano® 14 Margin B1	Ila
08057538	Soprano® 14 Margin B2	Ila
08057539	Soprano® 14 Margin B3	Ila
08057540	Soprano® 14 Margin B4	Ila
08057541	Soprano® 14 Opaque Dentine A0	Ila
08057542	Soprano® 14 Opaque Dentine A0	Ila
08057543	Soprano® 14 Opaque Dentine A1	Ila
08057544	Soprano® 14 Opaque Dentine A1	Ila
08057545	Soprano® 14 Opaque Dentine A2	Ila
08057546	Soprano® 14 Opaque Dentine A3	Ila
08057547	Soprano® 14 Opaque Dentine A3	Ila
08057548	Soprano® 14 Opaque Dentine A3,5	Ila
08057549	Soprano® 14 Opaque Dentine A3,5	Ila
08057550	Soprano® 14 Opaque Dentine A4	Ila
08057551	Soprano® 14 Opaque Dentine A4	Ila
08057552	Soprano® 14 Opaque Dentine B0	Ila
08057553	Soprano® 14 Opaque Dentine B0	Ila
08057554	Soprano® 14 Opaque Dentine B1	Ila
08057555	Soprano® 14 Opaque Dentine B1	Ila
08057556	Soprano® 14 Opaque Dentine B2	Ila
08057557	Soprano® 14 Opaque Dentine B2	Ila
08057558	Soprano® 14 Opaque Dentine B3	Ila
08057559	Soprano® 14 Opaque Dentine B3	Ila
08057560	Soprano® 14 Opaque Dentine B4	Ila
08057561	Soprano® 14 Opaque Dentine B4	Ila
08057562	Soprano® 14 Opaque Dentine orange	Ila
08057563	Soprano® 14 Opaque Dentine brown	Ila
08057564	Soprano® 14 Opaque Dentine yellow	Ila
08057565	Soprano® 14 Dentine A0	Ila
08057566	Soprano® 14 Dentine A0	Ila
08057567	Soprano® 14 Dentine A0	Ila
08057568	Soprano® 14 Dentine A1	Ila
08057569	Soprano® 14 Dentine A1	Ila
08057570	Soprano® 14 Dentine A1	Ila
08057571	Soprano® 14 Dentine A2	Ila

Device number	Product name	Class
08057572	Soprano® 14 Dentine A2	Ila
08057573	Soprano® 14 Dentine A3	Ila
08057574	Soprano® 14 Dentine A3	Ila
08057575	Soprano® 14 Dentine A3	Ila
08057576	Soprano® 14 Dentine A3,5	Ila
08057577	Soprano® 14 Dentine A3,5	Ila
08057578	Soprano® 14 Dentine A3,5	Ila
08057579	Soprano® 14 Dentine A4	Ila
08057580	Soprano® 14 Dentine A4	Ila
08057581	Soprano® 14 Dentine A4	Ila
08057582	Soprano® 14 Dentine B0	Ila
08057583	Soprano® 14 Dentine B0	Ila
08057584	Soprano® 14 Dentine B0	Ila
08057585	Soprano® 14 Dentine B1	Ila
08057586	Soprano® 14 Dentine B1	Ila
08057587	Soprano® 14 Dentine B1	Ila
08057588	Soprano® 14 Dentine B2	Ila
08057589	Soprano® 14 Dentine B2	Ila
08057590	Soprano® 14 Dentine B2	Ila
08057591	Soprano® 14 Dentine B3	Ila
08057592	Soprano® 14 Dentine B3	Ila
08057593	Soprano® 14 Dentine B3	Ila
08057594	Soprano® 14 Dentine B4	Ila
08057595	Soprano® 14 Dentine B4	Ila
08057596	Soprano® 14 Dentine B4	Ila
08057597	Soprano® 14 Enamel E0	Ila
08057598	Soprano® 14 Enamel E0	Ila
08057599	Soprano® 14 Enamel E0	Ila
08057600	Soprano® 14 Enamel E1	Ila
08057601	Soprano® 14 Enamel E1	Ila
08057602	Soprano® 14 Enamel E1	Ila
08057603	Soprano® 14 Enamel E2	Ila
08057604	Soprano® 14 Enamel E2	Ila
08057605	Soprano® 14 Enamel E3	Ila
08057606	Soprano® 14 Enamel E3	Ila
08057607	Soprano® 14 Enamel E3	Ila
08057608	Soprano® 14 Enamel E4	Ila
08057609	Soprano® 14 Enamel E4	Ila
08057610	Soprano® 14 Enamel E4	Ila
08057611	Soprano® 14 Enamel Clear	Ila
08057612	Soprano® 14 Enamel Clear	Ila
08057613	Soprano® 14 Enamel Clear	Ila
08057614	Soprano® 14 Opal Effect	Ila
08057615	Soprano® 14 Opal Intensive 1	Ila
08057616	Soprano® 14 Opal Intensive 1	Ila
08057617	Soprano® 14 Opal Intensive 2	Ila
08057618	Soprano® 14 Opal Intensive 2	Ila
08057619	Soprano® 14 Opal Intensive 3	Ila
08057620	Soprano® 14 Opal Intensive 3	Ila
08057621	Soprano® 14 Opal Intensive 4	Ila
08057622	Soprano® 14 Opal Intensive 4	Ila
08057623	Soprano® 14 Opal Clear	Ila
08057624	Soprano® 14 Opal Clear	Ila
08057625	Soprano® 14 Enamel Effect clear-white	Ila
08057626	Soprano® 14 Enamel Effect sun	Ila
08057627	Soprano® 14 Enamel Effect sun	Ila
08057628	Soprano® 14 Enamel Effect apricot	Ila
08057629	Soprano® 14 Enamel Effect apricot	Ila
08057630	Soprano® 14 Enamel Effect amber	Ila
08057631	Soprano® 14 Enamel Effect amber	Ila
08057632	Soprano® 14 Enamel Effect gray	Ila
08057633	Soprano® 14 Enamel Effect gray	Ila
08057634	Soprano® 14 Enamel Effect light-blue	Ila
08057635	Soprano® 14 Enamel Effect Enh. amber	Ila

Device number	Product name	Class
08057636	Soprano® 14 Enamel Effect Enh. amber	Ila
08057637	Soprano® 14 Enamel Effect Enh. dark-gray	Ila
08057638	Soprano® 14 Enamel Effect Enh. dark-gray	Ila
08057639	Soprano® 14 Enamel Effect Enh. pink	Ila
08057640	Soprano® 14 Enamel Effect Enh. pink	Ila
08057641	Soprano® 14 Enamel Effect Enh. blue	Ila
08057642	Soprano® 14 Enamel Effect Enh. blue	Ila
08057653	Soprano® 14 Enamel Effect Enh. orange	Ila
08057654	Soprano® 14 Enamel Effect Enh. orange	Ila
08057655	Soprano® 14 Mamelon mango	Ila
08057656	Soprano® 14 Mamelon mango	Ila
08057657	Soprano® 14 Mamelon cream	Ila
08057658	Soprano® 14 Cuspid	Ila
08057659	Soprano® 14 Cuspid	Ila
08057660	Soprano® 14 Fossa	Ila
08057661	Soprano® 14 Fossa	Ila
08057662	Soprano® 14 Repair Enamel	Ila
08057663	Soprano® 14 Repair Dentine A0	Ila
08057668	Soprano® 14 Shade C-D Kit	Ila
08057669	Soprano® 14 Paste Opaque C1	Ila
08057670	Soprano® 14 Paste Opaque C2	Ila
08057671	Soprano® 14 Paste Opaque C3	Ila
08057672	Soprano® 14 Paste Opaque C4	Ila
08057673	Soprano® 14 Paste Opaque D2	Ila
08057674	Soprano® 14 Paste Opaque D3	Ila
08057675	Soprano® 14 Paste Opaque D4	Ila
08057676	Soprano® 14 Margin C1	Ila
08057677	Soprano® 14 Margin C2	Ila
08057678	Soprano® 14 Margin C3	Ila
08057679	Soprano® 14 Margin C4	Ila
08057680	Soprano® 14 Margin D2	Ila
08057681	Soprano® 14 Margin D3	Ila
08057682	Soprano® 14 Margin D4	Ila
08057683	Soprano® 14 Opaque Dentine C1	Ila
08057684	Soprano® 14 Opaque Dentine C2	Ila
08057685	Soprano® 14 Opaque Dentine C3	Ila
08057686	Soprano® 14 Opaque Dentine C4	Ila
08057687	Soprano® 14 Opaque Dentine D2	Ila
08057688	Soprano® 14 Opaque Dentine D3	Ila
08057689	Soprano® 14 Opaque Dentine D4	Ila
08057690	Soprano® 14 Dentine C1	Ila
08057691	Soprano® 14 Dentine C1	Ila
08057692	Soprano® 14 Dentine C2	Ila
08057693	Soprano® 14 Dentine C2	Ila
08057694	Soprano® 14 Dentine C3	Ila
08057695	Soprano® 14 Dentine C3	Ila
08057696	Soprano® 14 Dentine C4	Ila
08057697	Soprano® 14 Dentine C4	Ila
08057698	Soprano® 14 Dentine D2	Ila
08057699	Soprano® 14 Dentine D2	Ila
08057700	Soprano® 14 Dentine D3	Ila
08057701	Soprano® 14 Dentine D3	Ila
08057702	Soprano® 14 Dentine D4	Ila
08057703	Soprano® 14 Dentine D4	Ila
08057705	Soprano® 14 Paste Opaque Gingiva bright	Ila
08057706	Soprano® 14 Paste Opaque Gingiva dark	Ila
08057715	Soprano® 14 Gingiva dark	Ila
08057716	Soprano® 14 Gingiva dark	Ila
08057717	Soprano® 14 Gingiva violet	Ila
08057718	Soprano® 14 Gingiva violet	Ila
08057725	Soprano® 14 Bleaching Kit	Ila
08057726	Soprano® 14 Paste Opaque A00	Ila
08057727	Soprano® 14 Paste Opaque B00	Ila
08057728	Soprano® 14 Margin A00	Ila

Device number	Product name	Class
08057729	Soprano® 14 Margin B00	Ila
08057730	Soprano® 14 Bleach Opaque Dentine	Ila
08057731	Soprano® 14 Bleach Opaque Dentine	Ila
08057732	Soprano® 14 Bleach Opaque Dentine	Ila
08057733	Soprano® 14 Bleach Dentine 1	Ila
08057734	Soprano® 14 Bleach Dentine 1	Ila
08057735	Soprano® 14 Bleach Dentine 1	Ila
08057736	Soprano® 14 Bleach Dentine 2	Ila
08057737	Soprano® 14 Bleach Dentine 2	Ila
08057738	Soprano® 14 Bleach Dentine 2	Ila
08057739	Soprano® 14 Bleach Enamel 1	Ila
08057740	Soprano® 14 Bleach Enamel 1	Ila
08057741	Soprano® 14 Bleach Enamel 1	Ila
08057742	Soprano® 14 Bleach Enamel 2	Ila
08057743	Soprano® 14 Bleach Enamel 2	Ila
08057744	Soprano® 14 Bleach Enamel 2	Ila
08057757	Soprano® 14 Bleach Dentine 4	Ila
08057758	Soprano® 14 Bleach Dentine 4	Ila
08057759	Soprano® 14 Bleach Dentine 4	Ila
08057785	Soprano® 14 Bleach Dentine 3	Ila
08057786	Soprano® 14 Bleach Dentine 3	Ila
08057787	Soprano® 14 Bleach Dentine 3	Ila
08057800	Soprano® 14 Bleach Transpa-white	Ila
08057801	Soprano® 14 Bleach Transpa-white	Ila
08057916	Soprano® 14 Bonder paste EMF Alloys	Ila
08057917	Soprano® 14 Paste Opaque A2	Ila
08057918	Soprano® 14 Margin A2	Ila
08057919	Soprano® 14 Opaque Dentine A2	Ila
08057920	Soprano® 14 Dentine A2	Ila
08057921	Soprano® 14 Enamel E2	Ila
08057922	Soprano® 14 Opal Effect	Ila
08057923	Soprano® 14 Enamel Effect clear-white	Ila
08057924	Soprano® 14 Enamel Effect light-blue	Ila
08057925	Soprano® 14 Mamelon cream	Ila
08058202	Soprano® 14 Gingiva light	Ila
08058203	Soprano® 14 Gingiva light	Ila
08058204	Soprano® 14 Gingiva orange	Ila
08058205	Soprano® 14 Gingiva orange	Ila
08058271	Soprano® 14 Powder Opaque Liquid	Ila
08057664	Soprano® 14 Opaque/Bonder Liquid	Ila
08055283	Soprano® Stains & Glaze Fluid	Ila
08055170	Livento® press Bleach 2 MT	Ila
08055171	Livento® press Bleach 3 MT	Ila
08058261	Soprano® 14 Powder Opaque A bright	Ila
08058262	Soprano® 14 Powder Opaque A dark	Ila
08058263	Soprano® 14 Powder Opaque B bright	Ila
08058264	Soprano® 14 Powder Opaque B dark	Ila
08058265	Soprano® 14 Powder Opaque C bright	Ila
08058266	Soprano® 14 Powder Opaque C dark	Ila
08058267	Soprano® 14 Powder Opaque D bright	Ila
08058268	Soprano® 14 Powder Opaque D dark	Ila
08058269	Soprano® 14 Powder Opaque A00	Ila
08058270	Soprano® 14 Powder Opaque B00	Ila
08055075	Livento® press Soprano® Stain & Glaze Kit	Ila